

SERO REPAIR

SeroLube Ocular Lubricant

Single Use Only Device

▲ **CAUTION:** Federal Law restricts the device to sale by or on the order of a veterinarian.

▲ **CAUTION:** The SeroRepair/SeroLube product is not provided sterile, but aseptically prepared. DO NOT use any component of the system if the packaging is opened or damaged.

COMPANY INFO: SeroLube is a product of SeroRepair LLC, a veterinary device and product manufacture based In Murray Utah. All SeroRepair products are produced in ISO 13485/cGMP (FDA human-use standard) facilities and adhere to those respective standards. While some components maybe be foreign-sourced, all critical components and contents are Made in the USA.

INDICATIONS FOR USE: SeroLube is a frozen blood serum that can be used as a natural lubricant on a variety of companion animal eye disorders. The anti-inflammatory properties and anti-collagenases in SeroLube mitigate corneal degradation in certain ulcers and stabilize the cornea scaffolding and framework. Blood serum eye drops are a beneficial part of treating other inflammatory conditions such as indolent ulcers, corneal trauma, allergic conjunctivitis, KCS (dry eye), and post operative care.


CONTRAINDICATIONS: The SeroLube product may be contraindicated when the patient may have a treatable ocular condition, but other indications would suggest not using an eye lubricant, or when the product packaging has been compromised. These blood products are not to be used for transfusion.

▲ **WARNING AND PRECAUTIONS:**

1. Do not use if the packaging is open or damaged.
2. Do not use if the product has had extensive temperatures exposure above 14f (-10c).
3. Do not use after expiration date.
4. Use only the Instruction for Use for the SeroLube product.
5. The companion animal owner should be informed of the risks associated with the use of any ocular lubricant and serums in general.

Instructions for Use

1. Properly restrain your companion animal.
2. Remove the cap from the SeroLube vial and place in a convenient location.
3. When restrained, position your hands to gently open the eyelids.
4. Place 1-2 drops in the center of the open eye, ensuring even distribution.
5. Repeat as instructed by your veterinarian, if indicated for the other eye.
6. Replace the cap and return the vial to refrigeration.
7. Additional dosing should be as instructed by your veterinarian.

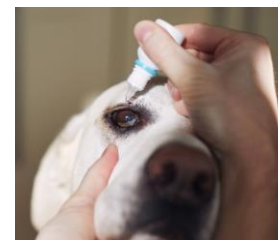



Discontinue Use for any of the following:

1. Your companion animal is having a severe adverse reaction following application of drops.
2. New or intensified redness appears.
3. New or excessive discharge from the eye/s being treated occurs.
4. Your companion animal is frequently rubbing at the treated eye/s.

Manufactured and distributed by:

SeroRepair, LLC
5650 S Green St.
Murray Utah 84123
(888) 885-6686
Email: info@serorepair.com
Site: www.serorepair.com

For Complaint Reporting: Please reference the SeroRepair LOT and REF number in all communications. Call or email SeroRepair for product questions, concerns, returns, or adverse events at 888-885-6686 or support@serorepair.com

COPYRIGHT © 2021-2024 SeroRepair LLC, ALL RIGHTS RESERVED.